

# Alles gurgelt!

## Gargle with Us!



1



Get information and register at [allesgurgelt.at](https://allesgurgelt.at). As soon as you have registered, you will receive a personalized barcode. By showing your barcode, you will receive **5 test sets once a month** at any BIPA outlet in Vienna.

2



Go to [allesgurgelt.at](https://allesgurgelt.at) and click on **"take your test now"**. Log in, follow the instructions in the WebApp and take your test.

3



**Confirm your identity** for the test by holding your passport, ID card or e-card towards the camera.

4



If you agree to a **video recording** of the test procedure, a **test certificate with an EU compliant QR code** will be issued for you with your negative test result, which you can use as entrance test for restaurants, gyms and events depending on current Covid regulations. If you skip this step, you will receive a test result, but no officially valid test certificate.

5



Now perform the **test**. You can gargle or rinse your mouth with the saline solution.

6



After performing the test, put the **sample** back into the LEAD Horizon packaging.


7



Now just hand in your test:

- **Mon-Sat:** at all **REWE outlets in Vienna** (BIPA, BILLA, BILLA PLUS, PENNY), and at **petrol stations in Vienna** (BP, SHELL, JET, OMV and ENI), and at **ÖBB train stations** (West, Meidling, Hauptbahnhof/Central Station)
- **Sundays and public holidays:** petrol stations, train stations Meidling & Hauptbahnhof/Central Station (pick-up at 9am and 2pm)

8

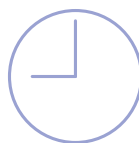

If you **hand in your sample before 9 a.m.**, you will receive your test result via email within a period of 24 hours.

9



You are **vaccinated**? Please still get tested regularly to protect yourself and others!  
You are **not vaccinated**? Get more information on [impfservice.wien](https://impfservice.wien) and get vaccinated to protect yourself and others!